

SleepAngel® medical products

Cleaning and care instructions for pillows

SleepAngel® products are ready for immediate use. The product is not sterilized.



1. Intended purpose/technology

SleepAngel® products are hermetically sealed:

- PureShield fabric protection: durable, soft touch, and high-performance textile blocks out contaminants. Vapor permeable and waterproof.
- SafeWeld seams: sealed seams provide complete barrier – no stitching holes.
- PneumaPure® filter: Breathable mechanical filter allows clean air flow into the pillow but blocks out pathogens and allergens.

2. Inspection & Cleaning

Before using the product:

- Clean the product (see above)
- Visually inspect the product's seams, cover/fabric and filters for damage or holes. A product with holes or damaged fabric or filter is contaminated and must be disposed of.

Everyday use:

- Visually inspect the product's seams, fabric and filters for damage or holes.
- A product with holes or damaged fabric or filter must be disposed of.
- Clean the pillow at least once a week, after each patient discharge or when soiled.
- Pillows need to be fluffed to maintain the fluffiness of the fibers.
- Ensure that all cleaning and disinfection agents in use comply with manufacturers guidelines.
- Clean with mild detergent and water solution.

- Wait to dry.
- The product is now ready to use

Note:

- Avoid abrasive scrubbing of the filter and fabric.
- Do not bleach, machine wash, dry clean, tumble dry, or iron the product.



The use of alcohol wipes and cleaning solutions can temporarily change the surface of the cover fabric.

3. Audit tool

Remove pillow from removable cover (if present) and check the following:

- Filter is not compromised
- Seams are intact
- Pillow is free from stains
- Pillow is free from visible tears or damage
- Pillow is free from unpleasant odor
- Pillow core is lofted and resilient

If any of the of the above statements is not true, the pillow has failed the audit and must be replaced.

4. Storage and instructions to read LOT

- The pillow can be safely stored flat or on side. Do not bend the pillow, this might damage the PneumaPure® filter.
- Do not place weights on the products.
- Store in a dry room at a temperature of +10...+30 °C.

Instructions to read LOT

Example LOT 11110722:

- 1111 – production number
- 07 – production month
- 22 – year

5. Disposal & Recycling

To dispose of the pillow, cut the pillow open and remove the inner pillow. Dispose of both in medical waste

Gabriel Scientific OÜ

Mauri tee 3, Papsaare küla, Pärnu linn 88317, Estonia

Tel: +372 56605967 www.SleepAngel-Medical.com info@SleepAngel-Medical.com

SleepAngel®

SleepAngel® medical products

Cleaning and care instructions for pillows

SleepAngel® products are ready for immediate use. The product is not sterilized.

Cleaning and disinfectant products that can be used:

- 1% Hydrogene Peroxide Solution
- Amonium Compounds
- Aniospray Quick
- Antifect extra
- AntiGone
- Bacillol AF
- Bacillol Foam
- Bactacid Wipes
- Chemisept wipes
- Chlorine Dioxide Solution, 10.000 ppm (lubatud kasutada tugeva määrdumise puhul, peale kasutamist puhastada üle niiske lapiga)
- Chlorine Dioxide Solution, 1000 ppm
- Chlorine Releasing Solution, 1000 ppm
- Clinell Alcohol Wipes
- Clinell Sporidical Wipe
- Clinell Universal Spray
- Cavi Cide Surface Decontaminant Cleaner
- Cavi wipes
- Descosept PUR Wipes RTU
- Dismozon® plus
- Ecopine, (5% Solution in Water)
- Incidin Foam & Incidin Plus
- Incidin OxyFoam S
- Incidin OxyWipes
- Incidin Pro
- Isopropyl Alcohol, %70
- Kohrsolin FF
- Life Clean
- Meliseptol Rapid
- Microbac Forte
- Mikrozyd AF Liquid
- Mikrozyd Sensitive Liquid
- Mikrozyd Sensitive wipes
- Oxivir Plus
- Perform 1% and 3%
- Phenol Solution, %1
- PDI Sani Cloth AF3 Wipes
- Pursept AF
- Rely+ON Perasafe
- Rely+ON Virkon
- Sani-Cloth Active

- Spray Off
- Sterisept wipes
- Super Sani Cloth Germicidal Disposable Wipe
- Terralin® protect (max. 4%)
- TPH Protect
- ULTRASOL® active
- Virusolve (5% & 10% Solution in Water)
- Wet Wipe

Note: After using cleaning agents containing chlorine, it is recommended to clean the surfaces with a damp cloth, because chlorine can damage the surface of the cover fabric of the product during long-term use and make it more sensitive to mechanical damage.

Cleaning and disinfection products that are not recommended for use:

- Disclean ENDO PAA (Peracetic Acid Solution)
- PDI Sani-Cloth Bleach Germicidal Disposable Wipe
- Peracetic Acid

This list is not exhaustive.

Strictly follow and do not exceed the concentrations provided in the instructions.



Gabriel Scientific OÜ

Mauri tee 3, Papsaare küla, Pärnu linn 88317, Estonia

Tel: +372 56605967 www.SleepAngel-Medical.com info@SleepAngel-Medical.com

SleepAngel®